



Product Description

PetSoft Home™ is a single patient multi-use instrument that can be used as a retractor/protector of tissue for exams and then be given to pet parent to use at home.

- Easily assess dental health and mouth injuries at home
- Safely monitor tartar buildup, gum color, and more as instructed by veterinarian

Intended Use

PetSoft Home was designed to be provided to pet owners by veterinarians to help with at home oral care and monitoring.

Directions

Supplied non-sterile.

- Remove instrument from the package.
- Bend neck to adjust desired position for specific application and begin retracting.

Cleaning Instructions

- After each use, rinse with water.
- Inspect device for visual damage.

! Bendable neck will weaken after repeated use.

! Over time and after repeated use, PetSoft Home will begin to show signs of normal wear. Visual changes like wear and tear or deterioration will indicate that the product should be replaced.



DentalAire Item Number	Description
DTP10973	PetSoft Home, 24 Pack for Home Care

Questions?



(800) 866-6881



info@dentalaire.com